DE

HINWEISE

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GB

NOTES

Nasara gitter tape (cross tape) is applied to specially defined areas of the skin and can help to relieve pain. Its effect is based on the stimulation of acupuncture points.

How to use

Apply the gitter tape 20-30 minutes before exercising, swimming, showering or bathing.

1. Thoroughly clean the skin at the points where the tape is to be applied and remove any grease

2. Fold in the broken edge of the backing film and pull it apart slightly. You can now easily remove the plaster from the backing film

3. To ensure that the plaster comes off as painlessly as possible, you should remove the hair from the area before applying it

4. Body heat activates the full adhesive strength of the gitter tape

5. After a short acclimatization period, the tape will have adapted optimally to the skin

Please discontinue use and remove the gitter tape if allergic reactions such as itching, redness, pain or blistering occur. Please consult a doctor if necessary.

Restriction

Do not apply the grid tape if the skin is not intact or has open wounds (or bleeding).

Do not use the tape on skin diseases and/or existing wounds.

Please contact the appropriate authority if a serious problem occurs.

- Non-sterile - For single use only
- Keep out of the reach of children
- Store in a cool place (10-30°C)
- Do not expose to direct sunlight
- Materials: polyester 80%, polyurethane 15%, acrylic adhesive, water-resistant

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